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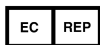
REV GA/2022-10

Ningbo Huakun Medical Equipment Co., Ltd.
No. 208 Binjiang Road
Cixi High-tech Industrial Development Zone
Cixi City
315301 Zhejiang, P.R. China

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GB Contents

1	General	Page	12
2	Safety and Responsibilities	Page	12
3	Modification of the product	Page	13
4	Before using the product	Page	13
5	Using the product	Page	14
5.1	Use with adults and children	Page	14
5.2	Use with infants and children	Page	14
5.3	Use with oxygen	Page	14
5.4	Use with ventilators	Page	14
6	Product Maintenance/Reprocessing	Page	15
7	Accessories/Package content	Page	15
8	Replacement parts and repairs	Page	15
9	Disposal	Page	15
10	Technical Specifications	Page	15
11	Explanation of symbols	Page	15/16

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1 General

- The product is used for emergency ventilation of patients and can be used by trained personnel as well as by laypersons.
- The following instructions must be followed by untrained users.
- The pocket mask constitutes a physical barrier between emergency responder and patient preventing the responder's lips from coming into contact with an unknown person.
- The product is mainly intended to be used in emergency care in hospitals and during first aid at the scene of accidents.
- These instructions reflect the state of the art at the time of publication.
- This product complies with the provisions of the EU Directive 93/42/EEC (Medical Device Directive) and Medical Device Regulation (EU) 2017/745.
- The instructions for use comply with DIN EN 82079-1.

2 Safety and Responsibilities

IMPORTANT! READ CAREFULLY BEFORE USE
KEEP FOR FUTURE REFERENCE

- Please observe the following instructions and care instructions before use.
- If in doubt about the use and connection of the product, please contact customer service.
- If possible, disposable gloves should be worn to prevent infection.
- If additional oxygen is administered care should be taken to ensure that smoking and other naked flames are forbidden in the vicinity of oxygen-carrying equipment and fittings.
- Keep the product free of oil and grease, as this could present an explosion hazard. This includes the exposure to cleaning alcohols, hand cream and adhesives – in contact with highly compressed oxygen explosive reactions may occur.
- In case of vomiting of the patient, clean the patient's airways and pocket ventilation mask. Should the valve mouthpiece and your skin become soiled, clean them directly. Then continue with ventilation immediately.
- **Single use products may not be reused!**
They must be disposed of immediately after use in accordance with chapter 9.
The reuse of single use products represents a potential risk for the patient or the user. It can result in contamination and/or reduced functionality.
- Contamination and/or limited functionality of the product can lead to injury, illness or death of the patient.

3 Modification of the product

- Apart from the described maintenance instructions, the product must not be modified or repaired by the operator!
- The manufacturer's responsibility is limited to the dimensional and physiological suitability of the product.
- Improper use, modifications to the product or negligence will void any warranty. This also applies to unauthorised repairs. Normal signs of wear and tear on the device are also excluded.

4 Before using the product

- Do not expose the product to direct sunlight or other light sources. Store and transport the product in a dry place in its original packaging at -30°C to $+50^{\circ}\text{C}$.
- Remove all delivered parts from the packaging. If necessary, remove the respective plastic covers and dispose of the packaging according to the current national regulations.
- Verify that parts or components do not show any damage, breakage or cracks. The product must also not be contaminated or soiled. If any of the conditions described above are present, the device must not be put into operation and must be disposed of in accordance with chapter 9.
- In case of doubt perform a comparison check with a brand-new product.
→ **Damaged products must not be used!**
After consulting customer service, the product must be disposed of according to chapter 9.
- Please note that the product is supplied in a non-sterile condition.
- The hygienic preparation of the product lies within the clinical responsibility of the user! The plastic product can be disinfected with a suitable cleaning agent (e.g. alcohol) before use. Sterilization is not permitted!

5 Using the product

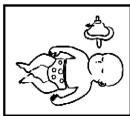
5.1 Use with adults and children

- Remove the mask from the case. If you have protective gloves, put them on.
- Press out the soft dome with your fingers and ensure that the one-way valve is correctly positioned.
- Remove any foreign objects from the patient's airways.
- Place the mask over the patient's mouth and nose according to the marking on the mask. Use the edge of the mask to press the patient's lower lip back to ensure that the patient's mouth remains open under the mask.
- Slowly blow twice into the ventilation valve and observe whether the patient's chest rises. If the patient's chest fails to rise, change the position of the patient's head and chin until you are able to apply ventilation properly, and a reaction of the chest can be seen.
- Ventilate the patient according to the current guidance for cardiopulmonary resuscitation.
- Continue ventilation until the patient shows a response or until you are relieved by next-level medical carer.

5.2 Use with infants and children

- The pocket ventilation mask can be rotated in order to fit the infant's/young child's face better.
- Place the nose section of the mask underneath the chin.
- Seal the pocket ventilation mask as described in the section for adults and children and ventilate the patient according to the current guidelines for cardiopulmonary resuscitation.


5.3 Use with oxygen

- The pocket ventilation mask can also be used with additional oxygen. Attach the oxygen tube to the oxygen valve (optional) and adjust the flow rate.

5.4 Use with other ventilators

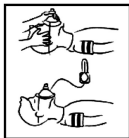- The pocket ventilation mask can also be used with ventilators. Simply remove the valve/filter system and attach the pocket ventilation mask to the ventilator.

6 Product maintenance/Reprocessing

- Single use products may not be reused!
- Please discard the product after use in accordance with the instructions in chapter 9.

7 Accessories/Package content

- Mask
- Protective case
- In case of compatibility questions please contact customer service.

8 Replacement parts and repairs

- Repairs must only be carried out by the manufacturer or by authorised persons.
- In case of functional or material problems, contact customer service immediately.
- If the cause of the error cannot be eliminated, the product must not be put into operation. After consulting the customer service, the product must be disposed of according to chapter 9 or returned to the sales distributor.

9 Disposal

- The product must be disposed of after use.
- Please note the expiry date if applicable.
The product must not be used after the expiry date!
- Safe disposal is carried out in accordance with current national regulations.

10 Technical Specifications

Operating temperature:	-18°C to +40°C
Inhalation resistance at a flow rate of 50 lpm:	<4.5 cm/H ₂ O
Exhalation resistance at a flow rate of 50 lpm:	4.5 cm/H ₂ O
Dimensions:	13 cm x 10.6 cm x 4.8 cm
Weight	approx. 117 g

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Observe the operating instructions

11 Symbolerklärung / Objaśnienie symboli / Explanation of symbols

	D	PL	GB
	Warnhinweis	Wskazówka ostrzegawcza	S Warning
	Hersteller	Producent	Manufacturer
	EU-Repräsentant	Przedstawiciel UE	EU Representative
	CE-Symbol	Symbol CE	CE symbol
	Zur einmaligen Verwendung / nur zur Verwendung an einem Patienten	Do jednorazowego użytku /tylko do użytku na jednym pacjencie	Single-use only / Only to be used on one patient
	Kein Öl oder Schmiermittel verwenden	Nie oliwić	Do not use any grease or other form of lubricant
	Nicht verwenden wenn Packung beschädigt ist	Nie użytkować, jeśli opakowanie jest uszkodzone	Do not use if package is damaged
	Importeur	Importer	Importer
	Medizinprodukt	Wyrób medyczny	Medical Device
	Produktionsdatum	Data produkcji	Date of manufacture
	Unique device identification	Unikalny identyfikator urządzenia (UDI)	Unique device identification
	Charge	Partia	Batch code
	Referenznummer	Numer referencyjny	Catalogue number
	Verfallsdatum	Data ważności	Use by date
	Menge in Stück	Liczba sztuk	Quantity in pieces